

# Safe- Life Pendant Docking Station

## A-BPLB-01 FAQ'S

**Q. How do I connect the pad to the monitor?**

**R. Simply plug the cable that looks like a landline telephone plug into the monitor**

**Q. Can I clean the Pad?**

**R. You cannot wash the pad where it is submerged in water, however you can wipe the pad with a cleaning solution and damp cloth.**

**Q. How do I store the pad when it is not in use?**

**R. Although you cannot fold or roll the pad, you can lay it flat. Protecting the sheet with a thick layer of cardboard is ideal.**

**Q. Can this be used for aged care if the patient goes to respite or hospital**

**R. Yes. There are some interconnection cables required but this could operate as a part of a nurse call system.**

### Technical Specifications:

- Simple to operate
- 760 x 510 mm
- 1 Year life expectancy
- Easy to clean maintenance
- For use with Cura1 BME
- weight range for reliable operation: 30 kg to 200 kg
- Made from soft PVC
- Vent holes which reduce the risk of moisture
- Replaceable Break cord

Smart Caller Safe Life  
5/ 270 Lower Dandenong Rd Mordialloc Vic  
3195  
PH: +61 3 9588 0833  
E: sales@safelife.com.au