

Hardwired Chair/ Bed Pad set up instructions

Items You will need:

1. Hardwired Chair/ Bed Pad *fig. 1a*
2. Hardwired pad transmitter *fig. 1b*




Step 2. Undo the screw of the battery compartment *fig 2a*

Step 3. Open battery compartment *fig 2b*

Step 4. Insert 3x AA Batteries *fig 2c*

Step 5. Adjust setting such as volume and tone now by turning the dial for volume control and moving the tone setting to either 1, 2 or 3. There is also a delay setting of 0 seconds, 2 seconds and 15 minutes, *fig 2d*

Optional power supply available.

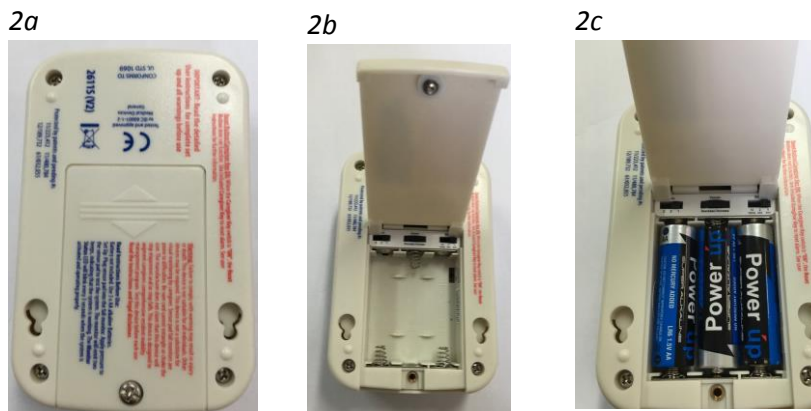

Step 6. Connect the cord telephone like plug into the monitor *fig 3d*

3d



Step 7. Test the pad. Place one hand on the pad.

Step 8. Remove your hand. The monitor will now sound an audible alarm. The Reset button LED and the Pad number will also flash. Press the reset button or tap care giver key to cancel the call.


Using the care giver key

The care giver key is a great way to ensure a resident does not attempt to tamper with the monitor. Fig 4a Rather than a carer pressing the reset button at an alarm, the care giver key is touched to the care giver key spot on the monitor *fig 4c*.

4a



4b

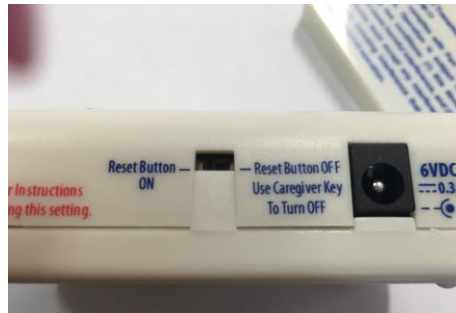

4c



Connecting to a nurse call system

Step 9. Connect the one end of the male to male cord into the monitor *fig. 5a and 5b*

5a



5b



Step 10. Connect other end directly into nurse call point or into the socket of a double adapter see 6a and 6b

6a



6b



Please note:

- Regular testing of this equipment is imperative in operation before a resident is placed onto the pad.
- All calls need to be cancelled from both the monitor via the reset button/ caregiver key and the nurse call point.
- Distance of transmission may vary according to building structure and radio signal interference.
- Battery life of the transmitter is approximately 12 months however battery life will be preserved as storage mode will occur when a pad is not in use for a period of time (approximately 3 minutes)
- Pads need to be kept flat, do not fold mats in storage or in transit.
- Pads are not to be immersed in water such as washing machines, dishwashers or aquatic recreational facilities.

For further information, please contact the Smart Caller sales team:

+61 3 9588 0833

sales@smartcaller.com.au