


Cordless Chair/ Bed Pad set up instructions

Items You will need:

1. Cordless Chair/ Bed Pad *fig. 1a*
2. Cordless pad transmitter *fig. 1b*
3. Cordless pad monitor (receiver) *fig. 1c*




- Step 1.** Turn the monitor to the back. *fig 2a*
Step 2. Undo the screw of the battery compartment *fig 2a*
Step 3. Open battery compartment *fig 2b*
Step 4. Insert 3x AA Batteries *fig 2c*
Step 5. Adjust setting such as volume ad tone now by turning the dial for volume control and moving the tone setting to either 1, 2 or 3. *fig 2d*
 Optional power supply available.

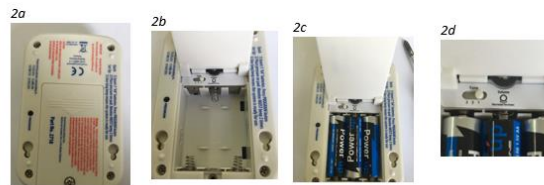

- Step 6.** Using a pen or screw driver hold down program button until monitor beeps 3 times. *fig 3a*



- Step 7.** Turn the monitor over to the front. Once you have pressed the program button the “Pad 1” Led will now illuminate green. *fig. 4a*
 Take the pad transmitter and turn the switch to the on position. *fig.4b*

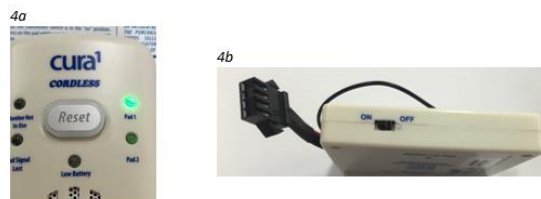

- Step 8.** Once you have turned on the transmitter into the on position the monitor will ask if there is a second transmitter by “Pad 2” LED illuminating. *fig 5a*



- Step 9.** If there is a second pad turn the second transmitter on.
Step 10. If no second pad is to be programmed, press the reset button.



Step 12. Close the press studs on the pocket (fig 6a).

Step 13. Test the pad. Place one hand on the pad.

Step 14. Remove your hand. The monitor will now sound an audible alarm. The Reset button LED and the Pad number will also flash. Press the reset button or tap care giver key to cancel the call.

Using the care giver key

The care giver key is a great way to ensure a resident does not attempt to tamper with the monitor.

Fig 7a Rather than a carer pressing the reset button at an alarm, the care giver key is touched to the care giver key spot on the monitor fig 7c.



Connecting to a nurse call system

Step 15. Connect the one end of the male to male cord into the monitor fig. 8a and 8b



Step 16. Connect other end directly into nurse call point or into the socket of a double adapter see 9a and 9b



Please note:

- Regular testing of this equipment is imperative in operation before a resident is placed onto the pad.
- All calls need to be cancelled from both the monitor via the reset button/ caregiver key and the nurse call point.
- Distance of transmission may vary according to building structure and radio signal interference.
- Battery life of the transmitter is approximately 12 months however battery life will be preserved as storage mode will occur when a pad is not in use for a period of time (approximately 3 minutes)
- Pads need to be kept flat, do not fold mats in storage or in transit.
- Pads are not to be immersed in water such as washing machines, dishwashers or aquatic recreational facilities.

For further information, please contact the Smart Caller sales team:

+61 3 9588 0833

sales@smartcaller.com.au