

CORDLESS PAD QUICK START GUIDE

Each pad will require a transmitter (A-CL433-TX) to pair with a CBM-02 cordless monitor



A-CBM-02



A-CCP-433



A-CBP-02-433M

Fig 1.0

Step 1:

Open battery compartment at the rear of the monitor and insert batteries (fig 1.0)

Or

Plug A-CBM/BME-PP into the monitor using the power socket (fig 1.1)



Step 2:

Connect transmitter(s) to the neck of the pad (fig 2.0) you wish to teach to the monitor.

Fig 2.0



Step 3:

Press and hold the program button on the back of the monitor (fig 3.0) down until it beeps 3 times.

Fig 3.0



Step 4:

Turn the On / Off switch (fig 4.0) on the side of the transmitter to the on position

Fig 4.0



Step 5:

Press the reset button in the middle of the monitor (fig 5.0) once.

Fig 5.0



Step 5:

Follow steps 1 – 5 to program all other transmitters to the monitor

SMART CALLER
Trading as SAFE- LIFE
5/ 270 LOWER DANDENONG RD MORDIALLOC VICTORIA 3195
03 9588 0833
03 9588 0933
Sales@safelife.com.au
www.safelife.com.au